

PLEASE READ CAREFULLY AND KEEP THIS PACKAGE
INSERT FOR FUTURE USE IN CASE YOU HAVE A PROBLEM.

**OPTI-
FREE**
Express
MULTI-PURPOSE
DISINFECTING
SOLUTION

LASTING COMFORT FORMULA
NO RUB™
ONE SOLUTION
REMOVES PROTEIN DAILY
CLEANS•RINSES•DISINFECTS•STORES

Directions for care of your lenses with OPTI-FREE® EXPRESS® MPDS
To clean, disinfect and remove protein from your contact lenses:

- Thoroughly rinse each side of the lens (5 seconds) with OPTI-FREE® EXPRESS® MPDS.
- Fill your ALCON® Lens Case with enough fresh OPTI-FREE® EXPRESS® MPDS to cover the lenses. Store lenses in the closed lens case overnight or at least 6 hours. After soaking, lenses are ready to wear.

If any debris remain on contact lenses, rinse with OPTI-FREE® EXPRESS® MPDS prior to insertion. You may leave your lenses in the unopened lens case containing OPTI-FREE® EXPRESS® MPDS for up to 30 days. After this time, your lenses must be cleaned and disinfected with OPTI-FREE® EXPRESS® MPDS every 30 days and prior to wear.

Your eyes are unique. Your eye care professional may recommend additional products or procedures to care for your lenses based upon your individual tear chemistry and lens-wearing schedule. Always follow your eye care professional's instructions. Do not change your directions for care of your lenses without consulting your eye care professional.

If your eye care professional directs you to rub your lenses, place several drops on the lens and rub for several seconds (up to 20). Then follow the directions above. If using SUPRALEN® Daily Protein Remover, follow those directions, including rinsing before putting the lenses on your eyes.

GOOD LENS CARE PRACTICES:

- Remove imprinted seal around bottleneck completely before use.
- Always wash and dry your hands before you handle your lenses.
- Clean, rinse and disinfect your lenses each time you remove them.
- Always handle the same lens, the right or the left, first in order to avoid mix-ups.
- After use, always empty and rinse the lens case with fresh OPTI-FREE® EXPRESS® MPDS and allow to air-dry.

DESCRIPTION:

OPTI-FREE® EXPRESS® Multi-Purpose Disinfecting Solution is a sterile, buffered, isotonic, aqueous solution containing sodium citrate, sodium chloride, boric acid, sorbitol, aminomethylpropanol, TETRONIC® 1304†, with edetate disodium 0.05%, POLYQUAD® (polyquaternium-1) 0.001% and ALDOX® (myristamidopropyl dimethylamine) 0.0005% preservatives.

ACTIONS:

When used as directed OPTI-FREE® EXPRESS® MPDS:

- Cleans by loosening and removing accumulations of protein and other deposits and debris from your lenses and continues to clean your lenses during disinfection and storage.
- Destroys harmful microorganisms (germs) on the surfaces of lenses.
- Is used to rinse and store your soft (hydrophilic) contact lenses.
- Is formulated for use in disinfection and storing lenses for up to 30 days after disinfection.
- It conditions the lens surface with a thin layer of moisture that allows it to remain more wetttable throughout the day.

No separate enzyme cleaner is required with OPTI-FREE® EXPRESS® MPDS when used daily.

No saline solution is required with OPTI-FREE® EXPRESS® MPDS.

COMFORT AND MOISTURE BENEFITS FOR LENS WEAR

While your lenses are soaking in the lens case, OPTI-FREE® EXPRESS® MPDS conditions the lens surface, leaving a hydrophilic layer on the lens. This maintains a shield of moisture between the lens and the eye, improving comfort by relieving the dryness and irritation associated with contact lens wear.

INDICATIONS (Uses):

For use in the daily cleaning, removing protein deposits, rinsing, chemical (not heat) disinfection, and storage of soft (hydrophilic) contact lenses (including silicone hydrogel lenses) as recommended by your eye care practitioner. OPTI-FREE® EXPRESS® MPDS can also be used with OPTI-FREE® SUPRALEN® Daily Protein Remover.

CONTRAINDICATIONS (Reasons Not to Use):

If you are allergic to any ingredient in this product, do not use.

WARNINGS:

PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care practitioner. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes, redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner. It is recommended that contact lens wearers see their eye care practitioner twice each year or, if directed, more frequently.

Re-use of solution or use of water with lenses may lead to contamination resulting in eye injury and potential loss of vision. See below for additional important safety information.

Important Safety Information:

- Always follow the product directions for use. Failure to follow product directions may lead to vision loss.
- Visit your eye care practitioner regularly.
- Always wash and dry hands before handling lenses.
- Do not use tap water, bottled water or saliva with lenses or lens case.

- Only use fresh solution to clean and disinfect contact lenses.
- Discard any remaining solution in your lens case after each disinfection cycle. (Never re-use solution.)
- Saline or rewetting drops will not disinfect your lenses.
- Always replace your solutions, lenses and lens case as directed.
- To avoid contamination, do not touch tip of container to any surface. Replace cap after using.
- Not for use with heat (thermal) disinfection.
- Store at room temperature.
- Use before the expiration date marked on the carton and bottle.
- Keep this and all medication out of the reach of children.

ADVERSE REACTIONS (Problems and what to do):

The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above:

IMMEDIATELY REMOVE YOUR LENSES.

- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, DO NOT put the lens back on your eye. Place the lens in the storage case and contact your eye care practitioner.
- If the lens has dirt, an eyelash, or other foreign body on it, or the problems stop and the lens appears undamaged, thoroughly clean, rinse and disinfect the lens, then reinsert it.
- If the problem continues, **IMMEDIATELY** remove the lens and consult your eye care practitioner.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

HOW SUPPLIED: OPTI-FREE® EXPRESS®

Multi-Purpose Disinfecting Solution is available for purchase in sterile 4 FL OZ (118 mL) and 10 FL OZ (300 mL) plastic bottles. Bottles and cartons are marked with a lot number and expiration date.

† TETRONIC® = Registered trademark of BASF.

© 2002-2004, 2006-2009 Alcon, Inc.

U.S. Patent Nos. 6,143,799; 6,365,636; 6,503,497; 5,393,491; 5,573,726; 5,631,005 and 6,319,464.

Questions or Comments?

Call 1-800-757-9195

MedInfo@AlconLabs.com

Manufactured by:

Alcon
Alcon Laboratories, Inc.
6201 South Freeway
Fort Worth, Texas 76134

9005078-1009